



AQUATRAM® ACCESS LIFT TRANSPORT CART

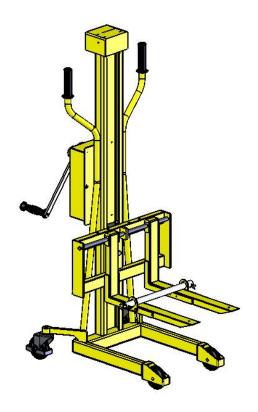


INSTALLATION AND USER'S GUIDE

IMPORTANT SAFETY INSTRUCTIONS READ AND FOLLOW ALL INSTRUCTIONS SAVE THESE INSTRUCTIONS


OPERATING INSTRUCTIONS: NOTE: READ THIS SECTION FULLY BEFORE USING THE TRANSPORT CART FOR THE FIRST TIME. BE SURE TO USE THE SUPPLIED COTTER-PINS TO HOLD THE LIFTING BAR IN PLACE.

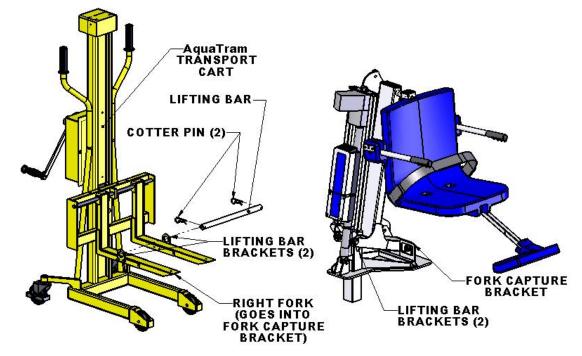

The AquaTRAM[®] Access Lift Transport Cart comes with a LIFTING BAR and two COTTER PINS, as shown above. The LIFTING BAR goes through the LIFTING BAR BRACKETS on the cart and the lift, and the COTTER PINS hold the bar in place. The right fork of the cart will also slide into the FORK CAPTURE BRACKET to keep the lift from tilting on the Transport Cart.

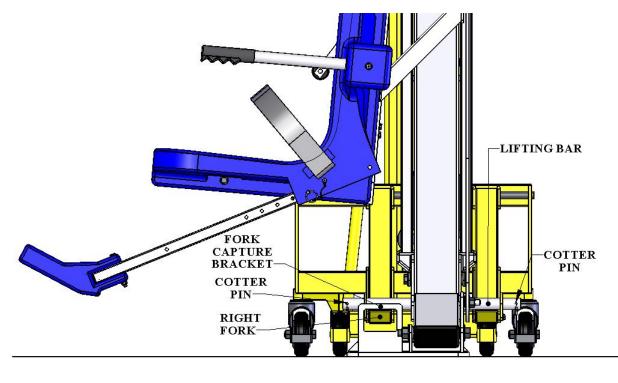

1

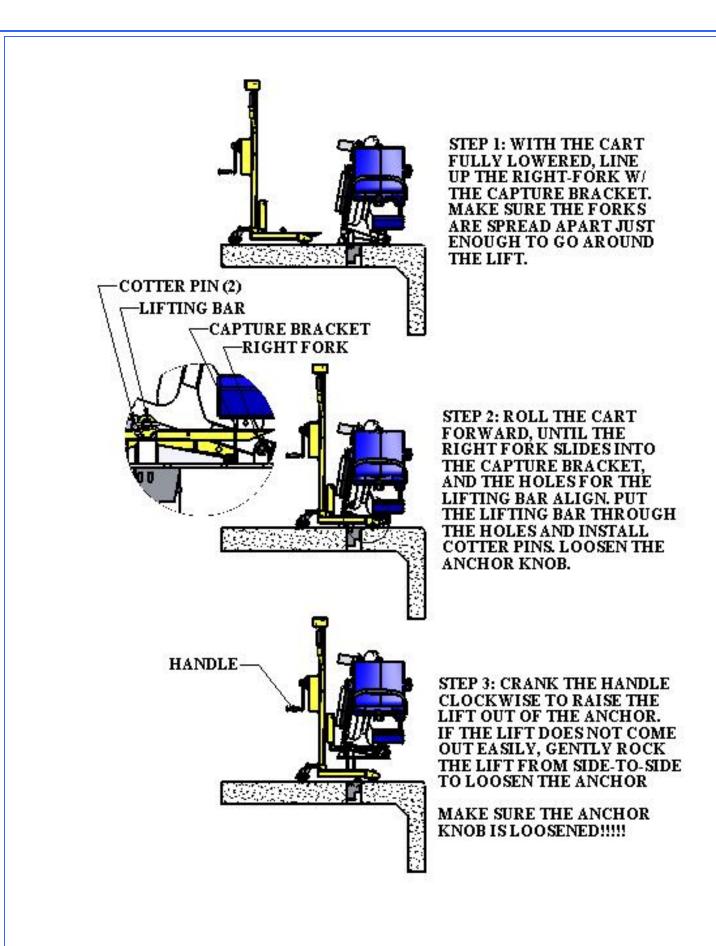



1620 HAWKINS AVE., SANFORD, NC 27330 • (919) 566-8000 10951 WEST LOS ANGLESE AVE., MOORPARK, CA 93021 • (805) 553-5000

All Pentair trademarks and logos are owned by Pentair or one of its global affiliates. Pentair Commercial Aquatics[™] and AquaTRAM[®] are trademarks and/or registered trademarks of Pentair Water Pool and Spa, Inc. and/or its affiliated companies in the United States and/or other countries. Unless expressly noted, names and brands of third parties that may be used in this document are not used to indicate an affiliation or endorsement between the owners of these names and brands and Pentair Water Pool and Spa, Inc. Those names and brands may be the trademarks or registered trademarks of those third parties. Because we are continuously improving our products and services, Pentair reserves the right to change specifications without prior notice. Pentair is an equal opportunity employer.

© 2014 Pentair Water Pool and Spa, Inc. All rights reserved.



P/N 11228 REV B 9/17/14